



SIZING CHART

CALL (877) TRACTION
(877) 872-2846

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
www.plastichorseshoes.com

Use the sizing chart to measure the required shoe size for your horse. The hoof edge should line up with the dotted line on the diagram.



Nails can be placed anywhere in the nail channel, so there is a little flexibility with the sizing. Order the size that most closely matches the hoof. Front & Hind patterns are the same.


The outside edge of the shoe (the outside line of the diagram) can be filed to bevel with the edge of the hoof. The back of the shoe can be trimmed away to fit the hoof.


If you order the wrong size by mistake, they can be exchanged, but we cannot return the cost of postage.



Make sure you print the pages at 100% size. Do not decrease or increase the paper. To test this, the box above should measure 1" in both directions.

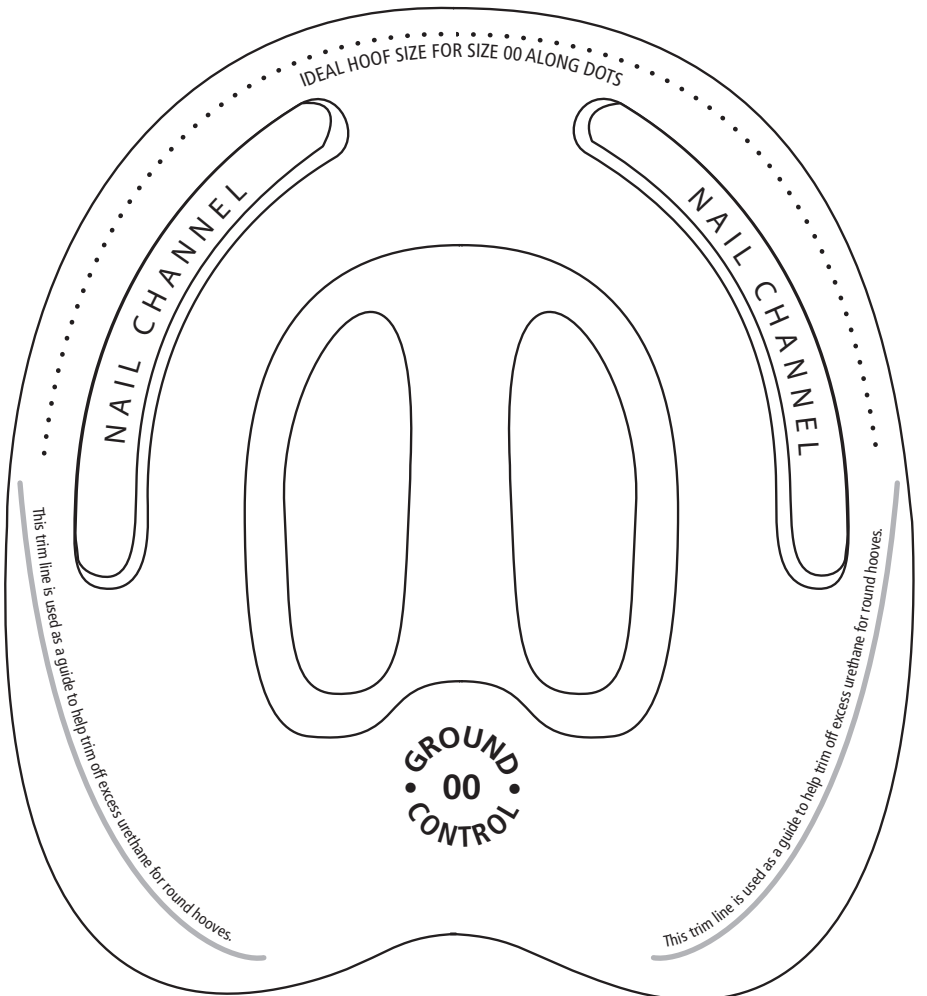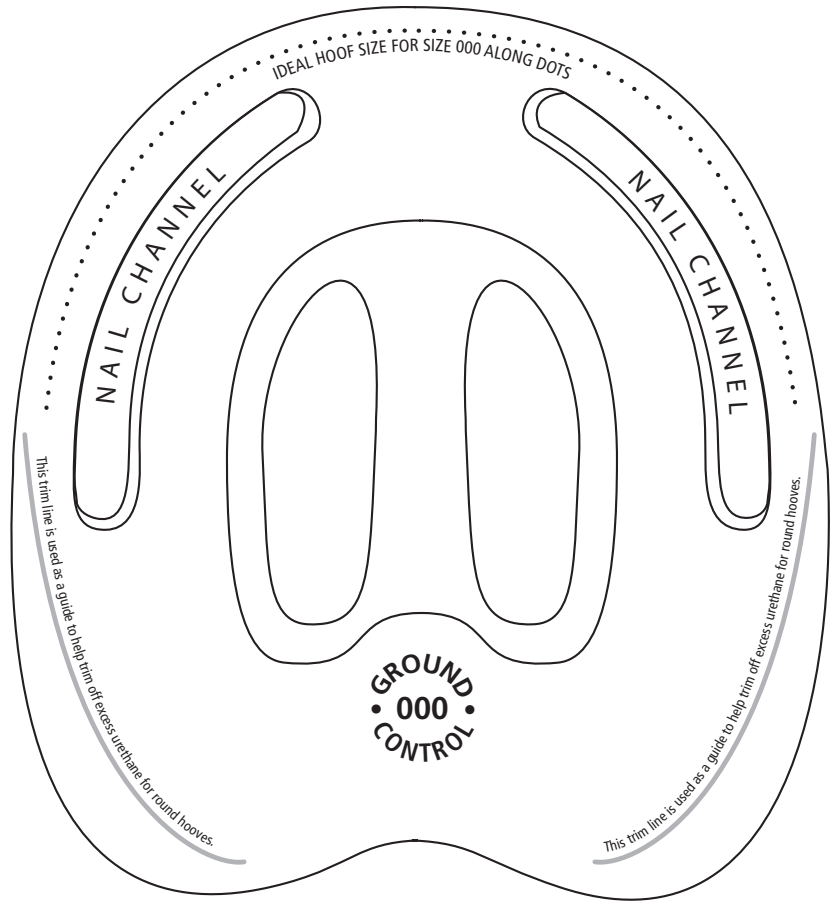

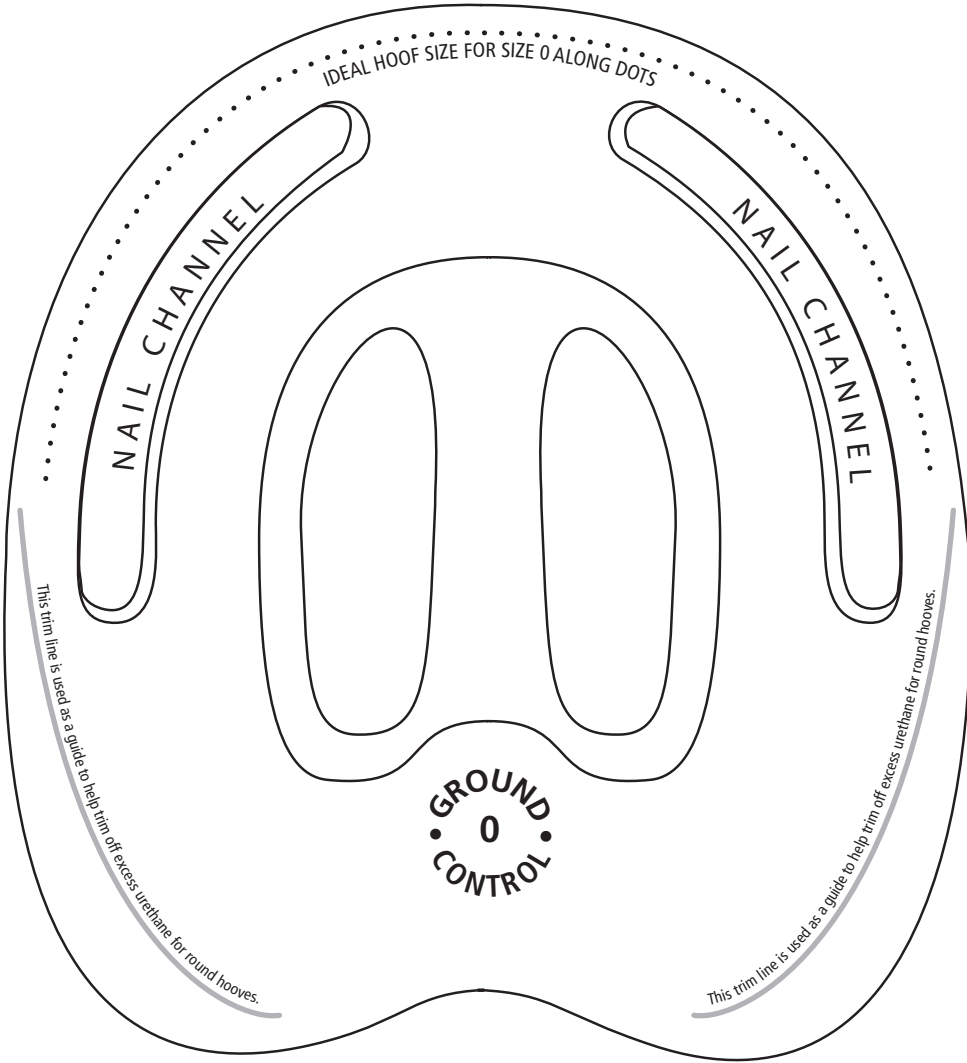

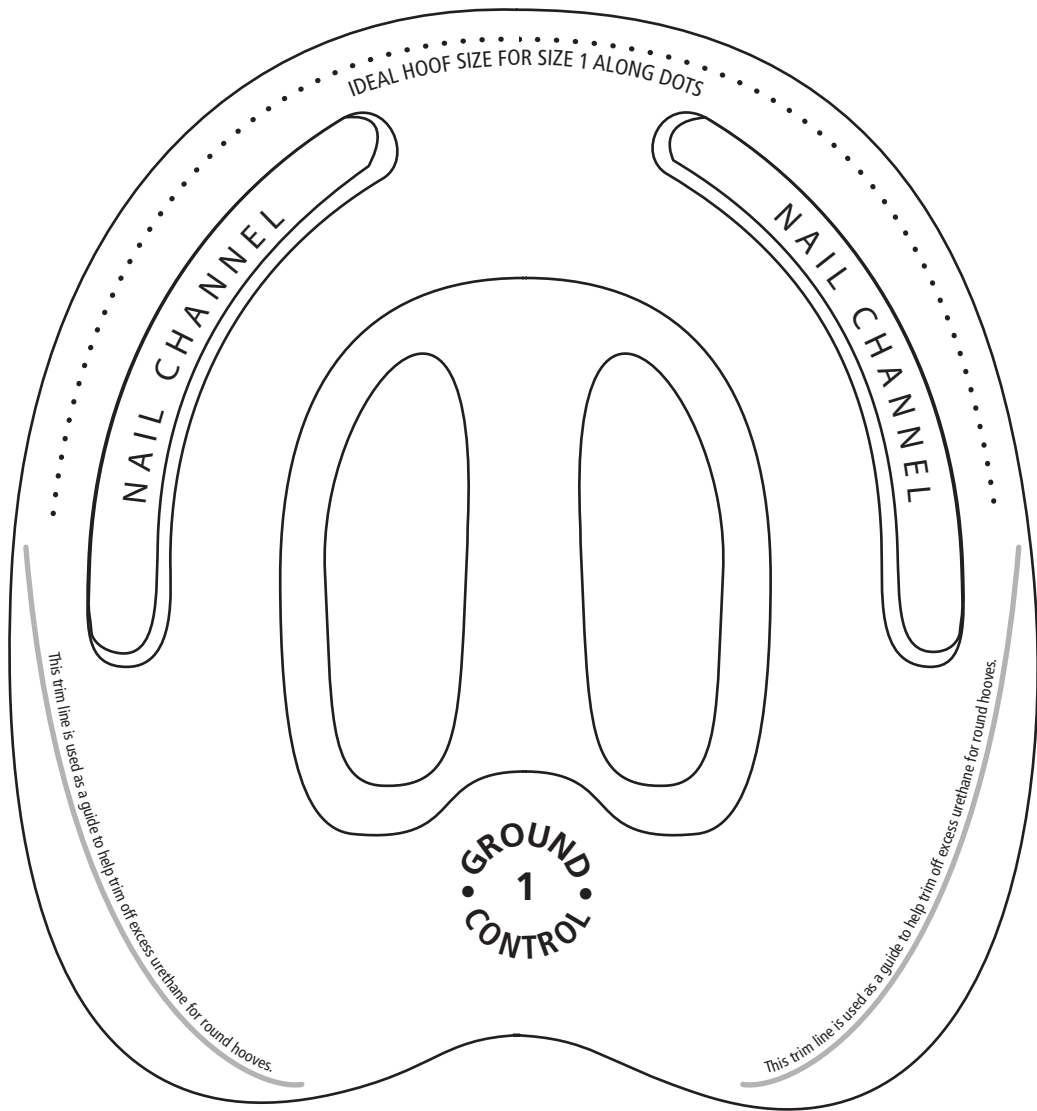

NAIL CHANNEL

NAIL CHANNEL

IDEAL HOOF SIZE FOR SIZE 1 ALONG DOTS

GROUND
1
CONTROL

This trim line is used as a guide to help trim off excess urethane for round hooves.

This trim line is used as a guide to help trim off excess urethane for round hooves.

